

HOMLAND



Walking Cane

Model: A1

Instruction Manual



11334-4

CONTENTS

Contents

Page

1. SAFETY GUIDELINES	2
2. SPECIFICATIONS	3
3. PARTS DESCRIPTION	3
4. PACKING LIST	4
5. ASSEMBLY INSTRUCTIONS	4
6. HEIGHT ADJUSTMENT	6
7. CARE AND MAINTENANCE	6
8. RECYCLING AND DISPOSAL	7
9. INSTALLATION VIDEO	7
10. WARRANTY	8
11. CONTACT US	8

1. Safety Guidelines

Warnings:

Misuse of the cane holder may lead to accidents or damage. Always follow the manufacturer's instructions for safe use.

If you are unable to understand these warnings, cautions, and instructions, contact us or a healthcare professional before attempting to use this equipment.


Contact HOMLAND at 888-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).

1. Make sure read and understand the user manual completely before assemble the product.
2. Before use, make sure the product is adjusted properly, and each part assembled correctly.
3. Make sure the push button must come out properly when adjust the foot tubes.
4. Please strictly follow the user manual when using the product. Do not use the product if any part is damaged.
5. Do not use strong corrosive cleaner on product surface.
6. Do not use the product on uneven ground or the product is not properly adjusted.
7. If the foot pad is worn or invalid,replace it in time.
8. Do not exceed the recommended weight capacity of the holder.
9. When moving with the cane, ensure the holder does not obstruct your motion or stability.
10. Do not hang items heavier than what the cane holder is designed to support.
11. Check the holder regularly for any signs of wear, damage, or loose parts.
12. Adjust the holder to the correct height or angle for comfortable access.
13. Place the cane with the holder on a stable, flat surface to prevent tipping.
14. Do not exceed the recommended weight capacity of the holder.
15. Ensure the cane holder is firmly attached to the cane or mobility aid before use.

2.Specifications

Max Weight Capacity	300 lb	Height Adjustable	40.35"-49.21"
Product size	9.05"(L) x 6.3"(W) x (40.35"-49.21")(H)	Package Size	9.05" (L)x 6.3" (W) x 20.47"(H)
Product Weight	1.76 lb	Package Weight	2.87 lb
Materials of Tube	Aluminum alloy		

3.Parts Description

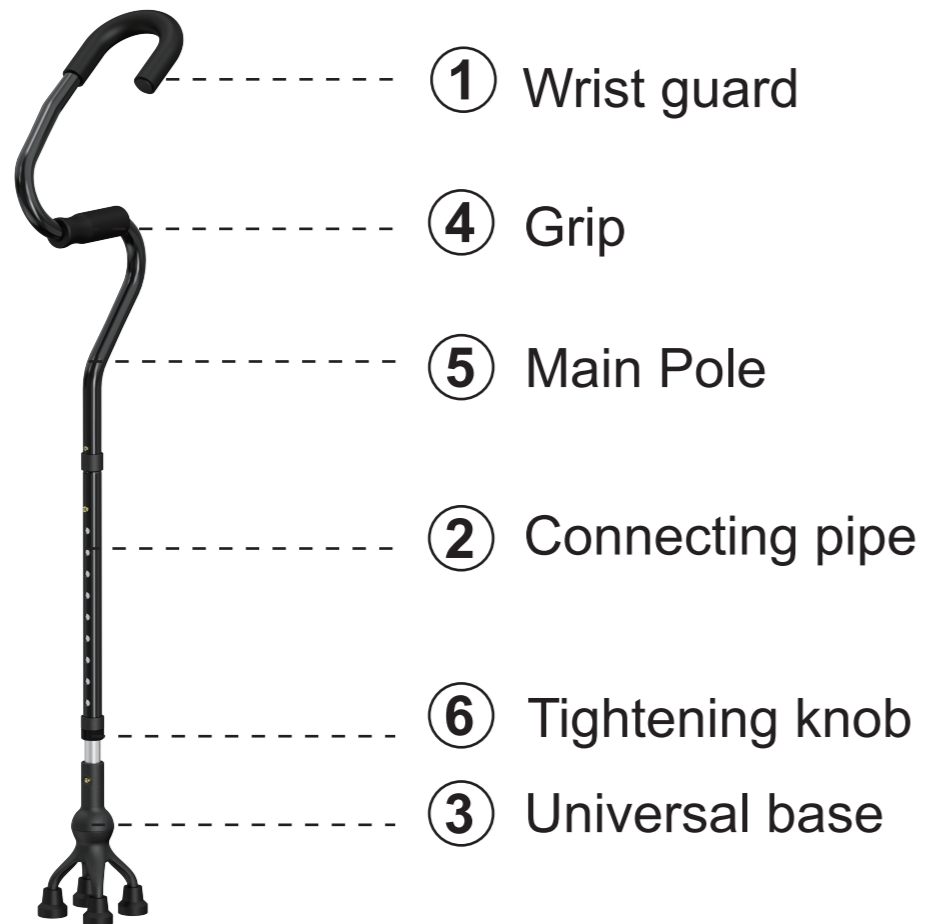

4.Packing list




- ① Main frame
- ② Connecting pipe
- ③ Universal base

5.Assembly Instructions




Step 1:
Install the connecting tube onto the universal base until the push button pops out completely.

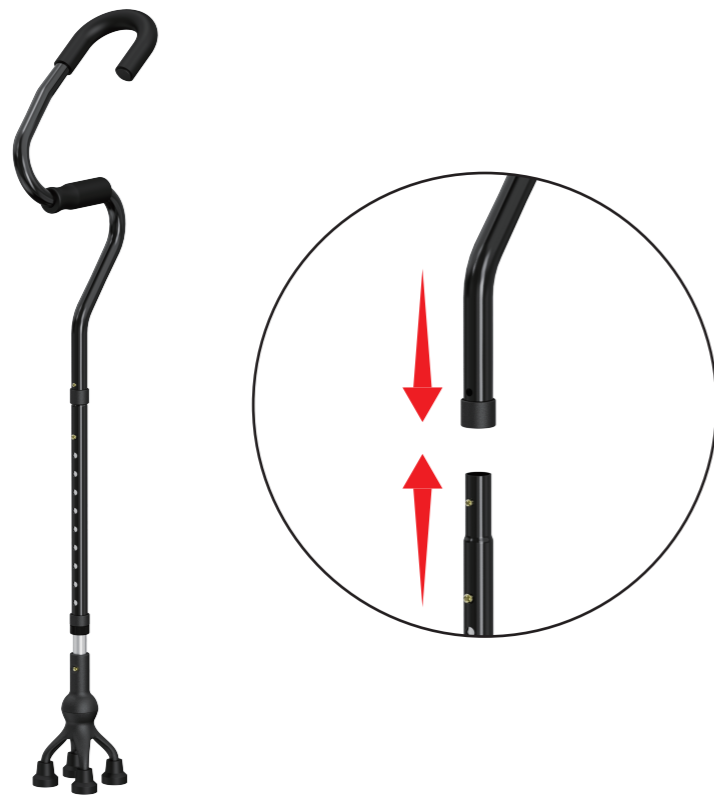

Step 2:

Install the main frame onto the connecting pipe until the push button pop out completely.

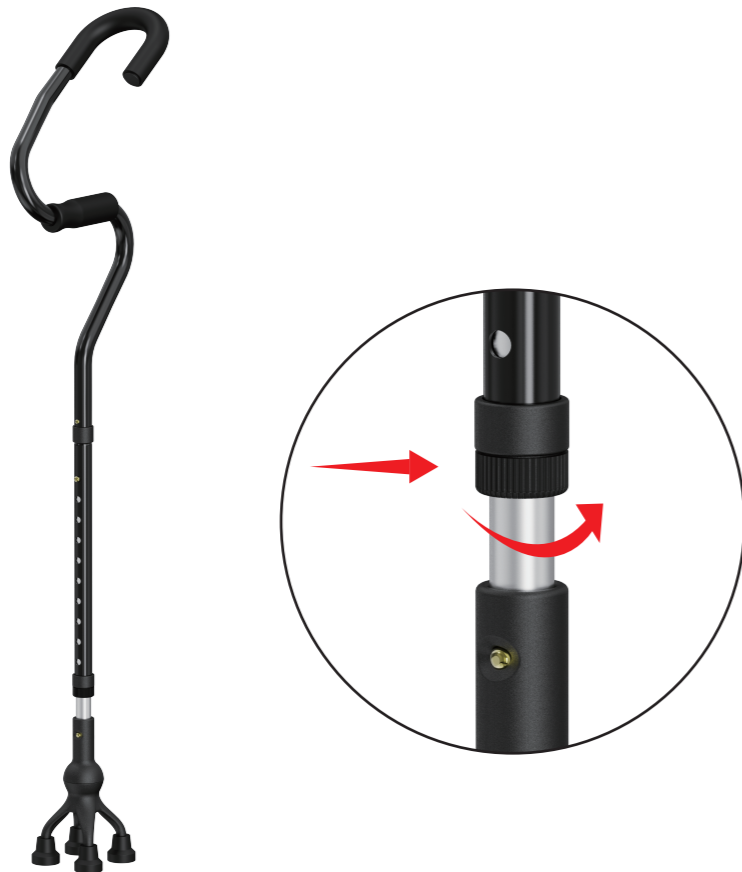

Step 3:

Adjust to the appropriate height to complete the installation. Please make sure the tightening knob is locked and the push button is fully pop out before use.

6.Height Adjustment



To adjust the height of the cane, squeeze the push button and adjust to the desired height until the push button pops out completely.

7.Care And Maintenance

The cane should be regularly inspected for signs of damage or wear and tear. Check for:

- Damage to tubing, including any bends or cracks at the tube joints.
- Worn or loose push buttons or elongated holes.
- Loose or worn handgrips.
- Smooth, worn or split tips. Clean the crutches occasionally using water and a mild detergent.

8. Recycling And Disposal

At the end of its useful life the aluminium and plastic components are to be separated and disposed of following the local recycling policy as these can be recovered and recycled.

9. Warranty



All HOMLAND products come with 1-year warranty from the date of purchase, scan QR code below or visit website <https://www.ihomland.com/pages/warranty-registration> for extra 1-YEAR WARRANTY registration

10. Installation Video



11. Contact Us

HONGKONG PISHON MEDICAL EQUIPMENT CO., LIMITED
UNIT 616, 6/F., KAM TEEM INDUSTRIAL BUILDING,
135 CONNAUGHT ROAD WEST,
SAI YING PUN, HONG KONG



1(888)-639-9558 (Mon-Fri 08:00 am-08:00 pm PST)



support@ihomland.com



www.ihomland.com