

17966

HOMLAND



Shower Chair

Model: B10

Instruction Manual

CONTENTS

Contents	Page
1. SAFETY GUIDELINES	2
2. SPECIFICATIONS	3
3. PARTS DESCRIPTION	3
4. PACKING LIST	4
5. ASSEMBLY INSTRUCTIONS	6
6. HEIGHT ADJUSTMENT	8
7. MAINTENANCE	9
8. WARRANTY	9
9. INSTALLATION VIDEO	10
10. CONTACT US	10
11. LIMITATION OF LIABILITY	11

1. Safety Guidelines

To ensure the safe installation, adjustment and operation of the shower chair, these instructions **MUST** be followed.


FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.

If you are unable to understand these warnings, cautions, and instructions, contact us or a healthcare professional before attempting to use this equipment.

Contact HOMLAND at 888-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).

Warnings:

Prior to each use, please make sure that the **Shower Chair** is firmly installed and the four supporting legs are on the same plane.

 **DO NOT** install the Shower Chair until you have read and understood the components and instructions. If you are unable to understand these instructions, please contact us directly before attempting installation. Otherwise, injury or damage may occur.

 **DO NOT** exceed the maximum user weight capacity of **350 lbs.**

 **DO NOT** stand on the shower chair.

 **DO NOT** twist or turn while seating on the shower chair, this may cause damage to the frame.

 **DO NOT** adjust the leg extensions while seated on the shower chair.

 **DO NOT** exceed the maximum user weight capacity of **350 lbs**

√ Check the chair for cracks or ruptures in the plastic. If you have concerns, contact HOMLAND at 888-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).

√ Check that all push buttons are fully protruded from the main frame through the leg extensions and are properly engaged.


√ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use

√ Users with limited physical ability should be supervised or assisted when using the Shower Chair.

2.Specifications

Max Weight Capacity	350 lb	Product Net Weight	9.04 lb
Product Size	18"(L)x15"(W) x26"-31"(H)	Package Size	17.3"(L)x13.3"(W) x6.3"(H)
Adjustable Height (Floor to seat)	14.7"-20"	Seat Size	17"(L)x12.5"(W) x1.5"(D)
Materials of Frame	Aluminum alloy	Materials of Seat	High Density Polyethylene

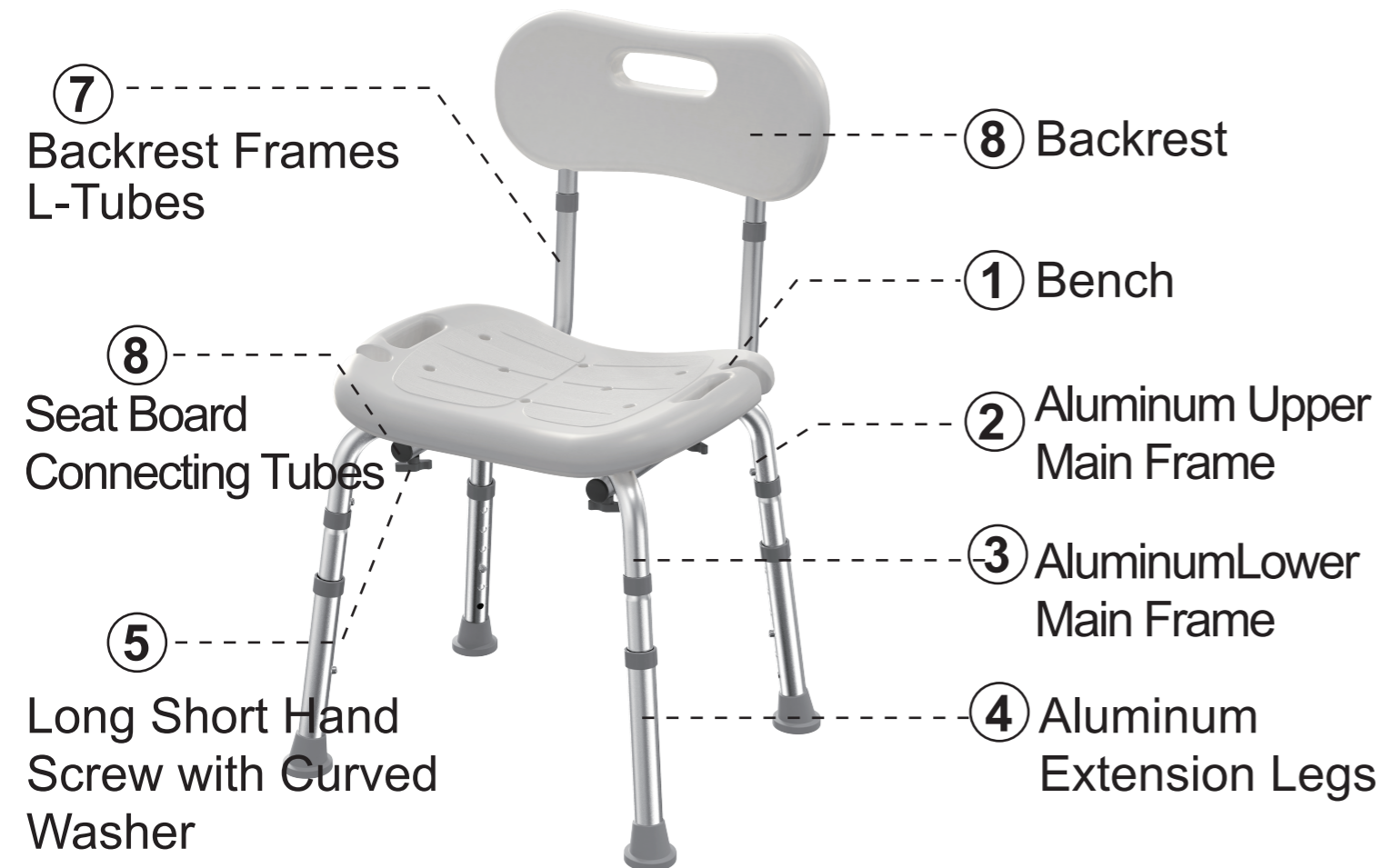


3.Parts Description




4.Packing List

NO	SKETCH	NAME	QTY
①		Bench	1pcs
②		Aluminum Upper Main Frame	1pcs
③		Aluminum Lower Main Frame	1pcs
④		Aluminum Extension Legs	4pcs
⑤		Long Short Hand Screw with Curved Washer	4pcs
⑥		Short Hand Screw with Curved Washer	1pcs


5. Assembly Instructions

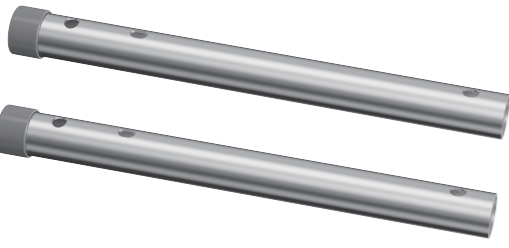7		Backrest Frames L-Tubes	2 pcs
8		Backrest	1 pcs
8		Seat Board Connecting Tubes	2 pcs


⚠ Ensure all parts are present as shown here. In the event of missing parts please contact us at 888-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).

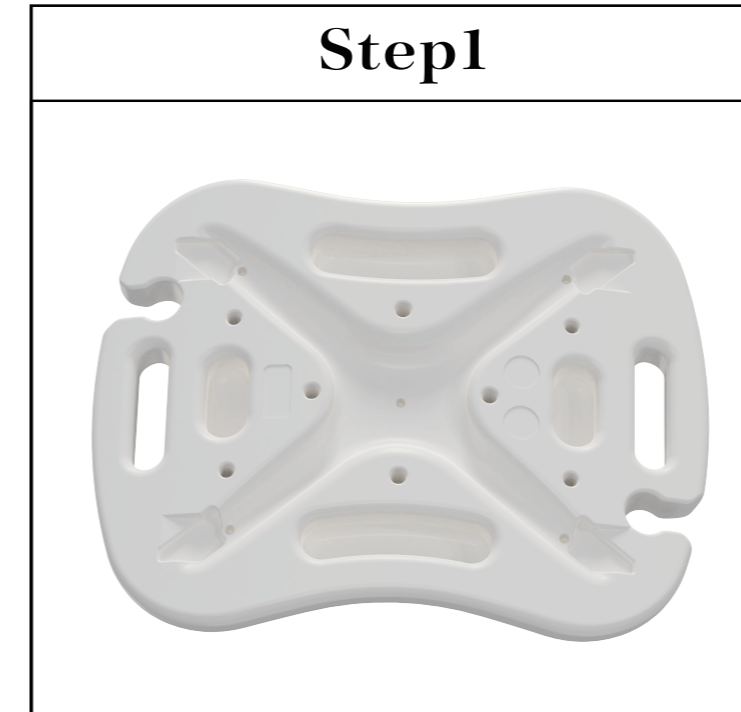


Place the seat plate upside down with the arrow markings facing backward.

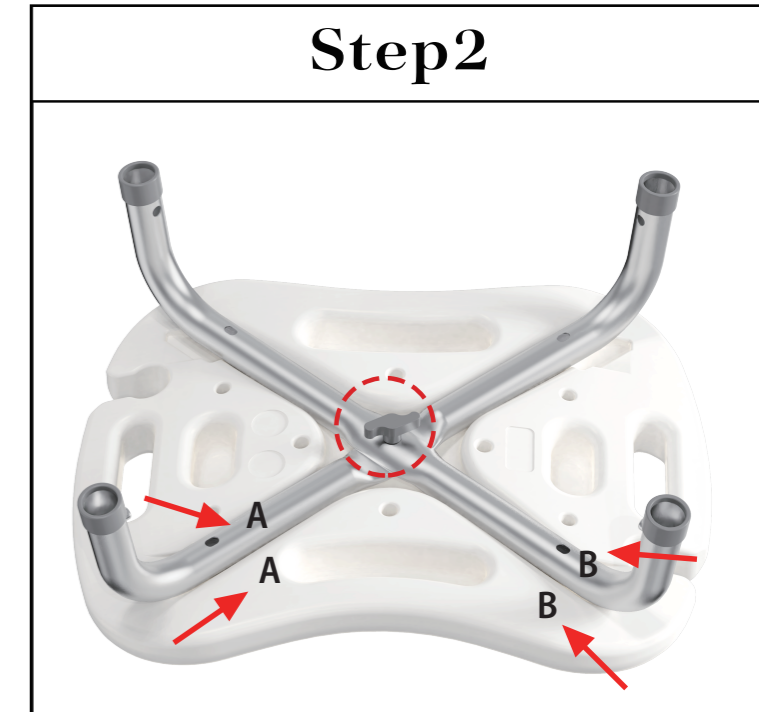

Insert the upper main frame (B) into slot B on the bottom of the seat plate. Then, insert the lower main frame (A) into slot A on the bottom of the seat plate. Once the frames are stacked, secure them in the middle using short hand screws and curved washers. (Note: The curved surface of the washer should align with the tubing.)

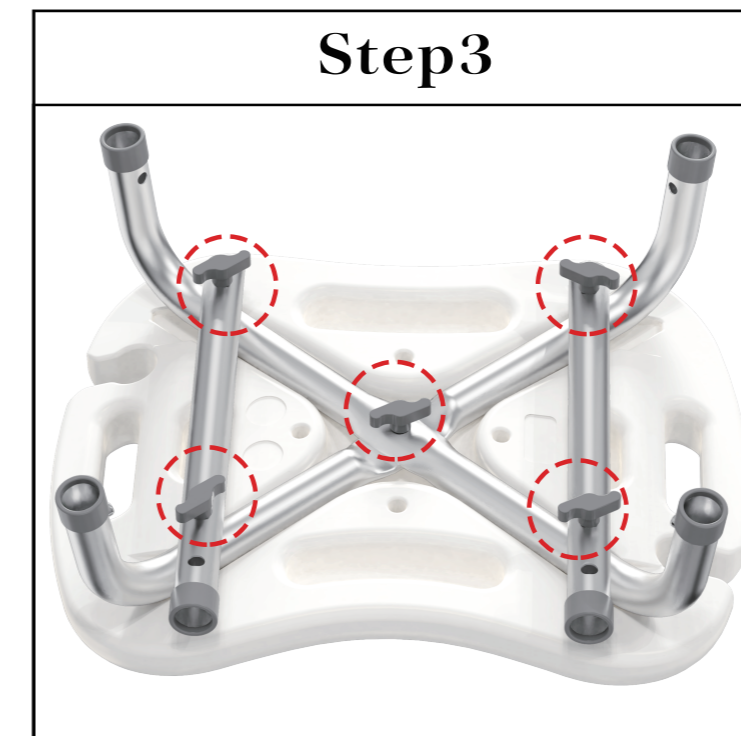

Attach the seat board connecting tubes to the main frame tubes and secure them using four long short hand screw.

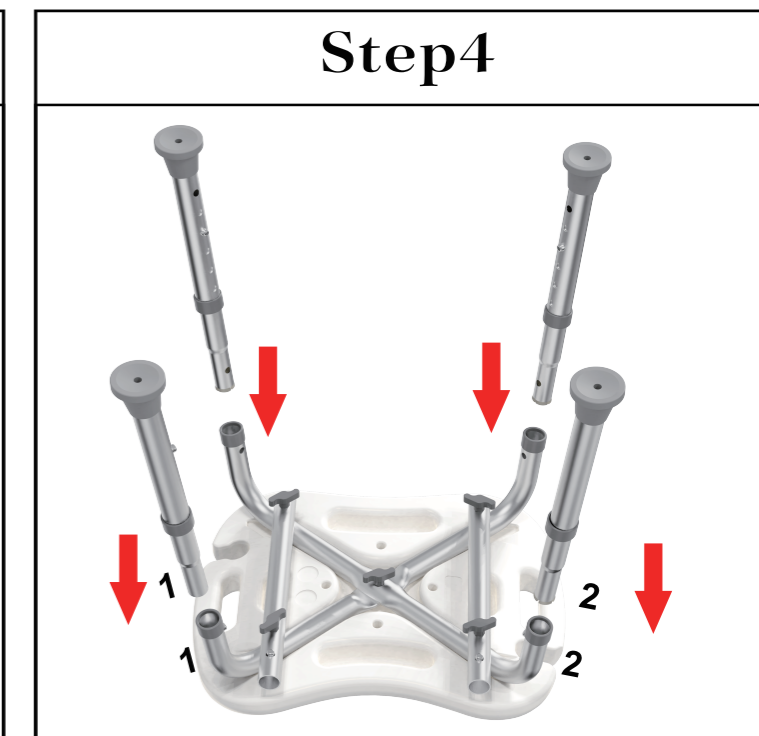

Assemble the four leg tubes to the main frame according to the numbered labels. (Note: Ensure that the snap buttons pop out through the holes in the leg tubes and that all four leg tubes are adjusted to the same height.)

Step5



Install the backrest frames L-Tube into the backrest until the ball bearing is fully ejected.

Step6




Insert the backrest to the back rest tubes. Ensure all push buttons are fully protruded from the main frame.

Step7



Installation Complete: Please ensure all knobs are securely tightened and the ball bearings are fully ejected before use.

How can we convert a shower chair with a backrest into a shower seat without a backrest?

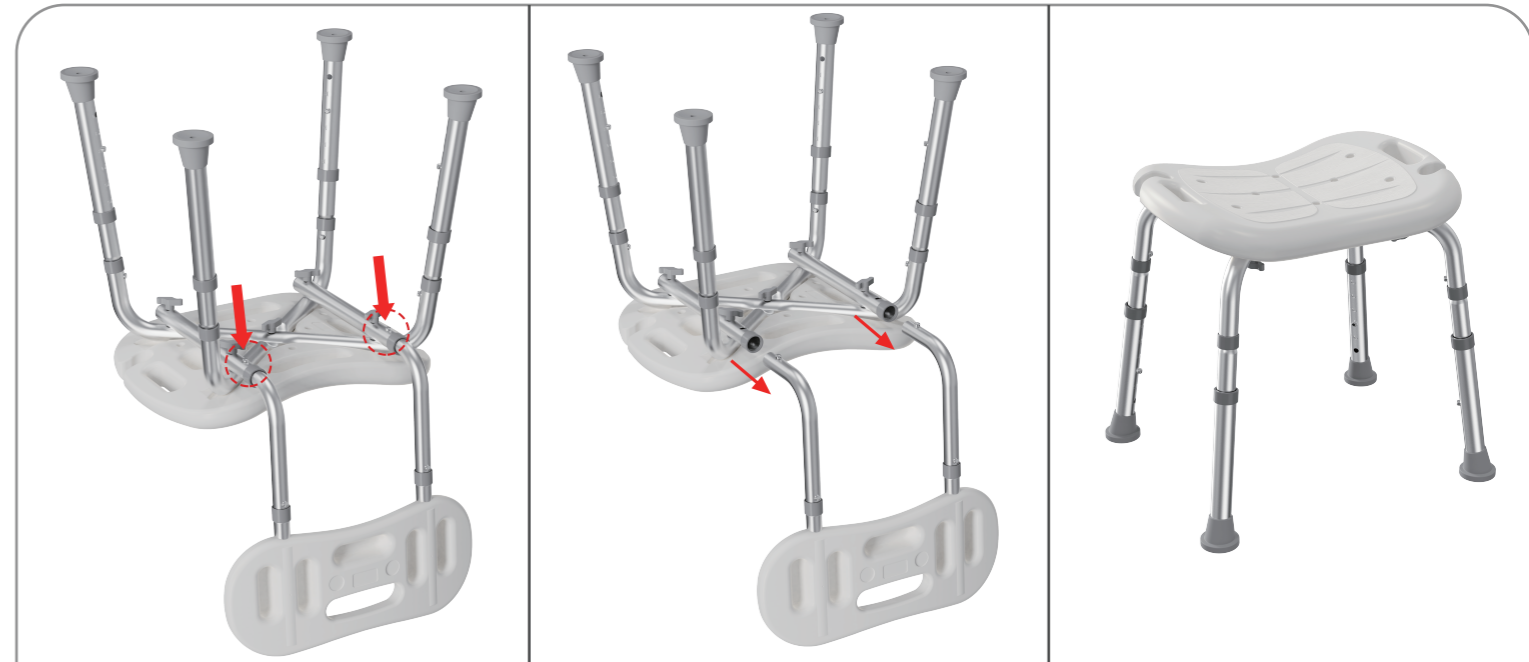

As shown in the illustration, press the button on the Backrest Frame L-Tubes and the Seat Board Connecting Tubes to separate the backrest from the main frame. The modification is now complete.

6.Height Adjustment



1. While standing in front of the assembled bench, as if preparing to sit, the back of the users knees should touch front side of the seat bench. This should allow the users feet to be firmly on the floor while properly seated.
2. Adjust the leg extensions as needed.
3. Ensure all four tips rest evenly on the floor.

7. Maintenance

Prior to use:

- Inspect the bath bench for damaged or loose components, and ensure that all parts are secure.
- Check that all four legs are adjusted evenly.
- Check the tips on the leg extension for rips, cracks or wear.
- Check the bench and back seat for cracks or ruptures in the plastic.
- Clean with regular household cleaning products, use a non-abrasive cleaner with warm water. DO NOT use abrasive powders or scouring pads.

8. Warranty



All HOMLAND products come with 1-year warranty from the date of purchase, scan QR code below or visit website <https://www.ihomland.com/pages/warranty-registration> for extra 1-YEAR WARRANTY registration

9. Installation Video



10. Contact Us

HONGKONG PISHON MEDICAL EQUIPMENT CO., LIMITED
UNIT 616, 6/F., KAM TEEM INDUSTRIAL BUILDING,
135 CONNAUGHT ROAD WEST,
SAI YING PUN, HONG KONG



1(888)-639-9558 (Mon-Fri 08:00 am-08:00 pm PST)

support@ihomland.com

www.ihomland.com

11. Limitation of Liability

Under no circumstances will the Warranty apply to any Product which has been used with unapproved components or to any Product which has been customized or modified, damaged, reused, or misused.

HOMLAND shall not be liable for a breach of the Warranty if: (i) Buyer makes any further use of such Defective Products after giving such notice; (ii) the defect arises because Buyer failed to follow HOMLAND's oral or written instructions as to the storage, installation, commissioning, use, or maintenance of the Products; (iii) Buyer alters, repairs, or conducts testing used to evaluate such Products without the prior written consent of HOMLAND or (iv) the defect is caused by the handling or packaging of the Products by Buyer.