

**17966**

THIS PRODUCT CONFORMS TO FDA, ISO 17966.

**HOMLAND**



**SHOWER CHAIR**

Model: SHC10

**Instruction Manual**

V3

# CONTENTS

## Contents

## Page

---

1. IMPORTANT SAFETY GUIDELINES	2
2. PARTS DESCRIPTION	3
3. PARTS LISTING	3
4. ASSEMBLY	5
5. ADJUSTING PROPER HEIGHT	7
6. MAINTENANCE	8
7. WARRANTY	8
8. CONTACT US	9

---

## 1 - SAFETY GUIDELINES

To ensure the safe installation, adjustment and operation of the shower chair, these instructions **MUST** be followed.


**FAILURE TO FOLLOW INSTRUCTIONS MAY RESULT IN SERIOUS INJURY TO THE USER OR DAMAGE TO THE PRODUCT.**

If you are unable to understand these warnings, cautions, and instructions, contact us or a healthcare professional before attempting to use this equipment.


Contact HOMLAND at 1(888)-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).


### Warnings:

Prior to each use, please make sure that the Shower Chair is firmly installed and the four supporting legs are on the same plane.


 **DO NOT** install the Shower Chair until you have read and understood the components and instructions. If you are unable to understand these instructions, please contact us directly before attempting installation. Otherwise, injury or damage may occur.

 **DO NOT** exceed the maximum user weight capacity of **400 lbs.**

 **DO NOT** stand on the shower chair.

 **DO NOT** twist or turn while seating on the shower chair, this may cause damage to the frame.

 **DO NOT** adjust the leg extensions while seated on the shower chair.

 **DO NOT** attempt to use the arms to supports full body weight, use the arm supports only for assistance.

 **DO NOT** exceed the maximum user weight capacity of **400 lbs.**


✓ Check the chair for cracks or ruptures in the plastic. If you have concerns, contact HOMLAND at 1(888)-639-9558.

✓ Check that all push buttons are fully protruded from the main frame through the leg extensions and are properly engaged.

✓ Ensure all four legs are adjusted evenly and are free of any obstructions prior to use

✓ Users with limited physical ability should be supervised or assisted when using the Shower Chair.

## 2 - PARTS DESCRIPTION

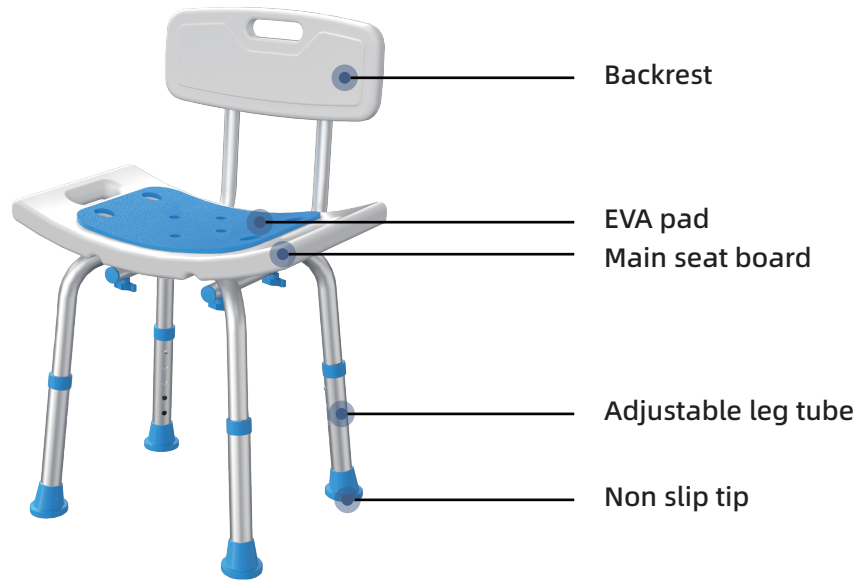

## 3 - PARTS LISTING

NO	SKETCH	NAME	QTY
1		Backrest tubes	2pcs
2		Foot tubes	4pcs

3		Square seat plate	1pcs
4		Upper main frame	1pcs
5		Lower main frame	1pcs
6		Backrest	1pcs
7		Short Hand Screw with Curved Washer	5pcs
8		Long Hand Screw with Curved Washer	4pcs

⚠ Ensure all parts are present as shown here. In the event of missing parts please contact us at 1(888)-639-9558 (Mon-Fri 08:00 am-08:00 pm PST).

## 4 - ASSEMBLY


### Step 1

Place the seat plate upside down with the arrow markings facing backward.



### Step 2

Insert the upper main frame (B) into slot B on the bottom of the seat plate. Then, insert the lower main frame (A) into slot A on the bottom of the seat plate. Once the frames are stacked, secure them in the middle using short hand screws and curved washers. (Note: The curved surface of the washer should align with the tubing.)

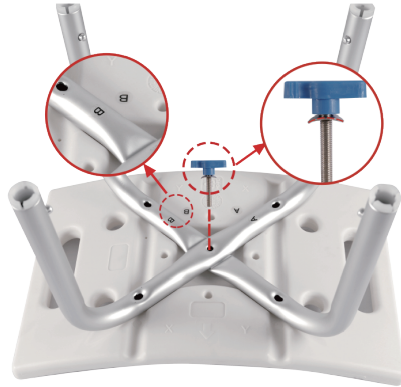


### Step 3

Attach the ends of the two backrest tubes to the backrest, then secure them with four short hand screws and curved washers. (Note: The curved surface of the washer should align with the tubing.)




### Step 4

Attach the assembled backrest to the assembled seat plate, securing it with four long hand screws and curved washers. (Note: The curved surface of the washer should align with the tubing.)

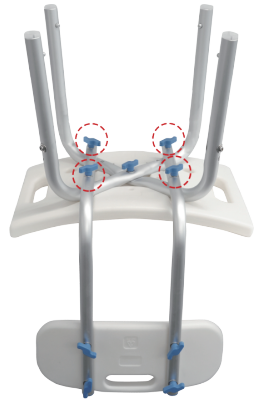

### Step 5

Assemble the four leg tubes to the main frame **according to the numbered labels**. (Note: Ensure that the snap buttons pop out through the holes in the leg tubes and that all four leg tubes are adjusted to the same height.)

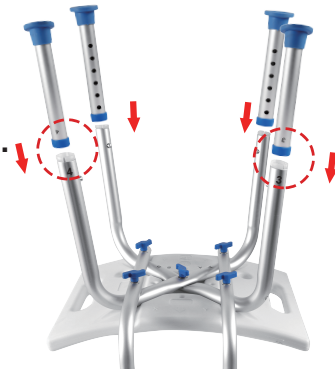

### Step 6

If desired, attach the blue seat cushion. The assembly is now complete.



## How can we convert a shower chair with a backrest into a shower seat without a backrest ?



1. Remove the chair backrest as shown, and keep the four long screws that were removed for safe keeping. (Do not remove the shorter screws in the middle.)



2. Then, remove the four short screws from the chair backrest.



3. To complete the installation, use the four short screws removed from the backrest to secure and ensure tightness between the two main leg tubes.

## 5 - ADJUSTING PROPER HEIGHT

1. While standing in front of the assembled bench, as if preparing to sit, the back of the users knees should touch front side of the seat bench. This should allow the users feet to be firmly on the floor while properly seated.
2. Adjust the leg extensions as needed.
3. Ensure all four tips rest evenly on the floor.

## 6 - MAINTENANCE

### • Prior to use:

- Inspect the bath bench for damaged or loose components, and ensure that all parts are secure.
- Check the tips on the leg extension for rips, cracks or wear.
- Always ensure that all screws are properly tightened, and that all four legs are evenly distributing the load.
- Remember that a higher seat will always be inherently less stable than a lower one.
- DO NOT tilt or rock the bench forwards or backwards while sitting.
- Clean with regular household cleaning products, use a non-abrasive cleaner with warm water. DO NOT use abrasive powders or scouring pads.

## 7 - WARRANTY



**All HOMLAND** products come with **1-YEAR WARRANTY** from the date of purchase. Scan QR code below or visit website for extra **1-YEAR WARRANTY** registration.  
<https://www.ihomland.com/pages/warranty-registration>

## Installation Video



## 8- CONTACT US

HONGKONG PISHON MEDICAL EQUIPMENT CO., LIMITED  
UNIT 616, 6/F., KAM TEEM INDUSTRIAL BUILDING,  
135 CONNAUGHT ROAD WEST,  
SAI YING PUN, HONG KONG



1(888)-639-9558 (Mon-Fri 08:00 am-08:00 pm PST)



support@ihomland.com



www.ihomland.com

## Limitation of Liability

Under no circumstances will the Warranty apply to any Product which has been used with unapproved components or to any Product which has been customized or modified, damaged, reused, or misused.

HOMLAND shall not be liable for a breach of the Warranty if: (i) Buyer makes any further use of such Defective Products after giving such notice; (ii) the defect arises because Buyer failed to follow HOMLAND's oral or written instructions as to the storage, installation, commissioning, use, or maintenance of the Products; (iii) Buyer alters, repairs, or conducts testing used to evaluate such Products without the prior written consent of HOMLAND or (iv) the defect is caused by the handling or packaging of the Products by Buyer.